

ERROR CODE	DESCRIPTION
E1	Switch on handpiece may be stuck in the ON position
E2	Footpedal switch may be stuck in the ON position
E3	Patient Return Electrode has poor connection
E4	Generator monopolar output error
E5	Handpiece has reached end of life
E6	Handpiece is not recognised as a Medtronic Advanced Energy device or failed test
E7	Generator bipolar output
E8	Monopolar output circuitry has exceeded a normal temperature
E9	Patient Return Electrode is not supported by current handpiece
E10	Handpiece must be Primed before performing Transcollation™



# Medtronic

## **Medtronic Australasia Pty Ltd**

2 Alma Road,  
Macquarie Park, NSW 2113  
Australia  
Toll Free: 1800 668 670

[medtronic.com.au](http://medtronic.com.au)  
[medtronic.co.nz](http://medtronic.co.nz)

## **Medtronic New Zealand Limited**

Level 3 – Building 5,  
Central Park Corporate Centre  
666 Great South Road Penrose,  
Auckland 1051  
New Zealand  
Toll Free: 0800 377 807

## **Medtronic New Zealand Limited**

via SCIONZ Authorised Sales Agent  
Unit A, 31 Cheshire Street,  
Parnell, Auckland  
New Zealand  
Toll Free: 0800 724 669

©2021 Medtronic Australasia Pty Ltd.  
All rights reserved. Printed in Australia.